# Instructions for Use

Brugsanvisning

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Bruksanvisning

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Manual

[elvie.com](http://elvie.com)

elvie pump

## Languages

English	4
Dansk	20
Deutsch	36
Español	52
Français	68
Italiana	84
Nederlands	100
Norsk	116
Svenska	132

## Instructions for Use

- Download the *Pump with Elvie* app to make use of the smart features: monitor milk volume in real time, track sessions and control the pump remotely
- The app is available on devices that support Bluetooth Low Energy. For the most up to date information on what devices the app can be used with, please visit [elvie.com/compatibility](http://elvie.com/compatibility).
- For other languages, visit [elvie.com/languages](http://elvie.com/languages)

### Questions?

For further assistance, please refer to the Help section in the *Pump with Elvie* app or visit [elvie.com/contact](http://elvie.com/contact). To find our Customer Care contact details for your region, please go to our website.

## 1. Intended use

- Express milk from breast

### 1.1 Indications for use

Powered breast pump, to express and collect milk from lactating women. The device is intended for a single user.

### 1.2 Operating life

- The operating life of the Hub is defined to be approximately 500 hours
- The expected operating life of the replaceable washable components is six months
- It is recommended to replace the washable components after long-term storage or before pumping for your next child

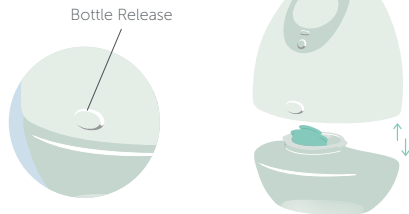
## 2. Getting started

### 2.1 Taking apart Elvie Pump


1. Press the Bottle Release to separate the Bottle from the Hub

3. Using the Spout, you can pour your expressed milk into a container of your choice for storage or feeding

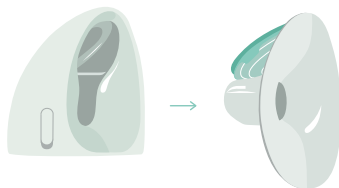5. Pull the Breast Shield away from the Hub to remove it


2. Remove the Valve from the Spout

4. Twist the Spout to remove it from the Bottle

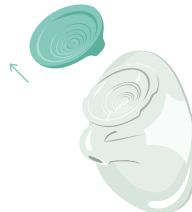6. Take the Seal off the Breast Shield

## 2.2 Cleaning before first use

Elvie Pump is not provided sterile. It must be cleaned and sterilized before first use. Always take apart components before cleaning or sterilizing.

### The Hub: electronics care

Do not get wet

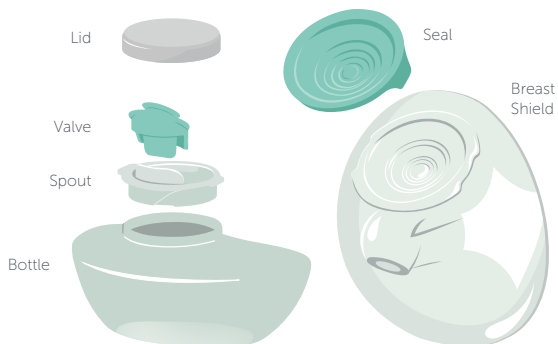
The Hub contains electronics that should be kept dry. A clean, damp cloth can be used to clean the Hub. The black semicircle on the base of the Hub monitors milk volume. Please keep this area dry and clear of residue. Any other cleaning method will be classed as misuse under the warranty (see section 4).

### Washable components

Cleaning washable components that come into contact with milk prevents the build-up of dry milk residue, which can cause the growth of bacteria that may be harmful to you and your baby.

## Before first use

### Clean washable components

- 1. Rinse** in cold, clear water (approx +20°C / +68°F)
- 2. Wash** in warm water (approx +30°C / +86°F) using mild dish washing soap and wipe afterwards with a clean cloth
- 3. Rinse** with cold, clear water (approx +20°C / +68°F) for 10 to 15 seconds

OR

Place on the top shelf of the dishwasher

### Sterilize washable components

Boil in water for five minutes

OR

Suitable for microwave, steam or cold water sterilizing methods.

We do not recommend microwave or steam sterilizing for more than five minutes\*

**Dry completely before assembly**

\*You should always refer to the manufacturers instructions and please be aware that some steam and microwave sterilizers reach high temperatures which may risk warping plastic parts. We do not recommend ultraviolet sterilisation.

## 2.3 Charging

Elvie Pump must be fully charged before first use and will not operate out of the box.

1. When charging, the Battery Light will be orange and Elvie Pump will not turn on.
2. When fully charged, the Battery Light will be green.
3. When the battery is low, the Battery Light will be red.

1. Battery charging

2. Fully charged

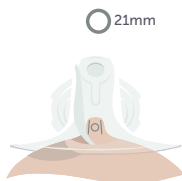
3. Battery low

## 2.4 Breast Shield sizing

Elvie Pump comes with Breast Shields in two sizes, so you can find a Breast Shield fit that is comfortable for you.


1. To determine the correct size, put the Breast Shield on with the sizing symbol facing upwards and place your nipple in the center of the fit lines. The fit lines should align with the outside of your nipple.
2. Try different Breast Shields to find the best fit for you.
3. To comfortably pump on both sides, repeat this process for both nipples to ensure you have the correct Breast Shield for each nipple.

Breast Shield too small

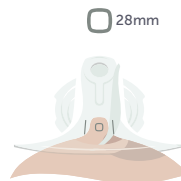Breast Shield correct fit

Breast Shield too big


Align the Breast Shield to the direction of your nipples when you are putting it on:

- Bend forwards and hold your breast from underneath
- Bring the Breast Shield to your breast, paying attention to the direction of your nipple
- You can use the markings on the Breast Shield to help align your nipple
- For further sizing guidance including use of Elvie Nipple Cushion please visit our website.


## 2.5 Assembling Elvie Pump

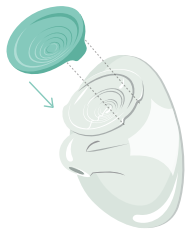1. Make sure all parts are dry before putting them together

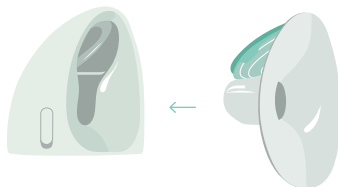
2. Push the Breast Shield into the Hub

3. Twist and click the Spout onto the Bottle

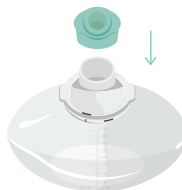4. Push the Valve onto the Spout

5. Click the Bottle onto the Hub

## 3. Using Elvie Pump

### 3.1 Turning on Elvie Pump

Once fully assembled, press Power to switch Elvie Pump on. Play / Pause will flash.

### 3.2 Finding the right compression

Elvie Pump is designed to be worn with your standard nursing bra and you can use it in either a sitting or standing position. Getting the right compression could impact milk volumes. Elvie Pump needs to be firmly pressed against your breast, there should be no gaps between Elvie Pump and your breast. Every bra has a different level of support, so make sure to get the compression right before each session. If you are using a particularly structured and inflexible bra, the Bra Adjuster may be used to create more room. The Elvie logo on the Bra Adjuster should face outwards.

**Tip: The Bra Adjusters are machine washable. To order additional Bra Adjusters, please visit [elvie.com](http://elvie.com) or an authorized retailer.**

### 3.3 Starting your session

Once Elvie Pump is turned on and positioned in your bra, press Play / Pause to start pumping.

### 3.4 Intensity settings and modes

#### Modes

- Elvie Pump has two modes: Stimulation and Expression. Elvie Pump will always start in Stimulation mode. This is a faster pumping rhythm to stimulate milk let-down
- Elvie Pump will automatically switch to Expression mode either when it detects milk has started to flow or after approximately two minutes

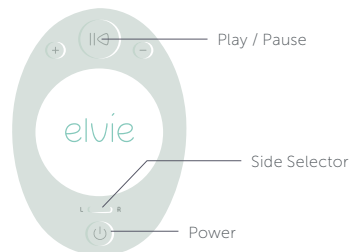
#### Switching sides

If you are switching Elvie Pump from one breast to the other, make sure you press the Side Selector while Elvie Pump is paused. When you restart Elvie Pump on your other breast, it will start in Stimulation mode

#### Intensity settings

- Elvie Pump has seven intensity settings, giving you control over what feels comfortable and works most efficiently in both Stimulation and Expression modes
- Once you can feel the suction on your nipple, use + - to adjust intensity to find your preferred setting in both modes. When you reach the maximum or minimum intensity setting, Play/Pause will flash three times

Note: Please visit [elvie.com](http://elvie.com) for the full list of automatic functions.


### 3.5 Changing the Bottle

Elvie Pump will automatically pause when the Bottle is full, indicated by the Bottle Light.

You must empty the Bottle before resuming your session.

Press the Bottle Release to separate the full Bottle from the Hub.

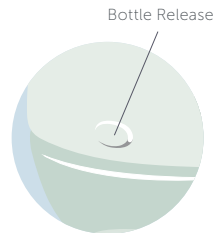Replace the full Bottle with an empty Bottle or pour your expressed milk into a container of your choice and continue with the same Bottle. Click the empty Bottle on to the Hub.

### 3.6 Ending your session

1. Press Play / Pause to stop pumping.
2. Unclip your bra and use your finger to break the seal around your nipple to avoid nipple damage.
3. Remove Elvie Pump from your bra.
4. Press Power to switch off.

### 3.7 Cleaning

Clean washable components after each use as advised.

### 3.8 Storing your milk

You may store your expressed milk in the Bottle using the Lid provided (both are fridge and freezer safe). Always make sure you remove the Bottle from the Hub before storing.

### 3.9 How to store Elvie Pump

Between sessions, please store your Elvie Pump out of direct sunlight and ensure the washable parts are dry. Keep out of reach of children. Recharge immediately after finishing your pumping session if the Battery Light is on.

### Long-term storage

To preserve the lifespan of your Elvie Pump, charge the Elvie Pump using the provided Charging Cable and store out of direct sunlight. Repeat this every 6 months as required. When removing from storage, plug the Charging Cable into your Elvie Pump and fully charge for approximately two hours.

The washable parts will require cleaning and drying before long-term storage and should be stored out of direct sunlight to avoid discoloration.

## 4. Warranty

Elvie Pump is backed by a limited manufacturer's warranty against any defects in materials and/or workmanship: the Hub for two (2) years and the washable components for 90 (ninety) days.

For the complete terms and warranty information, please visit [elvie.com/warranty](https://elvie.com/warranty) or contact our Customer Care team, visit [elvie.com/contact](https://elvie.com/contact). This policy is offered in addition to your legal rights.

This is a single-user product. Use by more than one person may present a health risk and voids the warranty. This product may only be returned for a refund within 30 days of purchase provided it is new, unopened, unused and in its original packaging with the manufacturer's factory seal still intact.

**Please keep your proof of purchase in the event that you need to contact customer care.**

**Residents in the United States: to request a hard copy of the US limited warranty information, call us at 508 300-9989, or write us at Elvie - Chiaro Technology, Office Manager, 184 7th Ave, Unit #107, New York, NY 10011.**

## UK PSTI STATEMENT OF COMPLIANCE

Chiaro Technology Ltd, 63-66 Hatton Garden, London, EC1N 8LE United Kingdom, hereby declares under its sole responsibility that the products outlined below:

Product Name: Elvie Pump Single and Double.

Model Number(s): EP01-01, EP01-02.

Batch: All serial numbers

Meet the applicable security requirements in Schedule 1 of the Product Security and Telecommunications Infrastructure (Security Requirements for Relevant Connectable Products) Regulation 2023. Support Period: 2 years from date of manufacture or 31 Dec 2025, whichever is later.

Signed for and on behalf of:

Legal Manufacturer: Chiaro Technology Ltd, 63-66 Hatton Garden, London, EC1N 8LE

Place of issue: Bristol

Date of issue: 01 Feb 2024

*Alastair Traquair*

Name: Alastair Traquair

Position: Director of Quality and Regulatory Compliance

## 5. Product specifications

### 5.1 Technical specifications

Vacuum range:	40 to 220 mmHg
Power:	5v DC, 3A max
Battery capacity and type:	1460mAh Lithium polymer
Size:	128mm x 110mm x 68mm / 5in x 4.3in x 2.7in
Weight:	225g / 7.9oz
Bottle:	Bottle volume markings are for indication only and should not be used for measuring.
Bluetooth info:	Bluetooth v5 (Bluetooth Low Energy). Range up to 10m
Operation temperature:	+5 to +30°C / +41 to +86°F
Transport/storage temperature:	-20 to +60°C / -4 to +140°F
Operation humidity:	15 to 90%
Transport/storage humidity:	15 to 90%
Ambient pressure:	90 to 106 kPa
Infrared details:	Peak frequency: 319THz, Radiant power: 55mW

### 5.2 Transport, storage and disposal

#### Transport

While traveling, use the Charging Cable provided and pair with the appropriate power source for your current location. Please check with the airline prior to travel for advice on carrying and using Elvie Pump while flying.

Temperature: -20 to +60°C / -4 to +140°F

Relative humidity: 15 to 90%

#### Storage

Temperature: -20 to +60°C / -4 to +140°F

Relative humidity: 15 to 90%

Ambient pressure: 90 to 106 kPa

#### Disposal

Elvie Pump contains a rechargeable lithium polymer battery. Please dispose of the Hub at a designated collection point for the recycling of waste electrical and electronic equipment. The appropriate disposal of Elvie Pump will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste for recycling, please contact your local authority or where you purchased your product.

13

### 5.3 EMC advice

#### **Declaration for Electromagnetic Compatibility (EMC):** For information use only.

This product conforms to the Radio Equipment Directive 2014/53/EU. For further information on device certification please visit [www.elvie.com/certification](http://www.elvie.com/certification).

**FCC Statement:** This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the equipment and receiver
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected
- Consult the dealer or an experienced radio/TV technician for help

**Stacking:** This device should not be used adjacent to or stacked with other equipment. And if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.

**FCC Radiation Exposure Statement:** This device should not be used adjacent to or stacked with other equipment. And if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.

**Caution:** The user is cautioned that changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Manufactured and Distributed by Chiaro Technology Limited

Model number: EP01

Document number: AW-000562

Revision: E

## 6. Safety

### 6.1 Warnings and precautions

- This is a single-user product. Use by more than one person may present a health risk and voids the warranty
- Elvie Pump operates within the 2.402-2.480GHz unlicensed ISM frequency band(s) and is limited to a maximum of 3dBm output power transmitted in the frequency band(s) in which the radio equipment operates
- It is recommended you consult with a licensed healthcare professional before using Elvie Pump if you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, implantable cardioverter defibrillator)
- Inspect for damage, clean and sterilize washable components that come into contact with your breast and breast milk prior to first use
- Clean all parts that come into contact with your breast and breast milk, and inspect for damage after every use
- During normal use, areas of the Elvie Pump may warm up. If your pumping environment is at the upper operating temperature limit (+30°C / +86°F) for the maximum session time (40 minutes), the temperature of the Breast Shield may reach up to +43°C / +109°F
- When Elvie Pump is pumping it will automatically stop after 40 minutes
- It is recommended to leave Elvie Pump for 60 minutes before using it again
- The black semicircle on the base of the Hub uses infrared (IR) energy to monitor the milk volume in the bottle when Elvie Pump is pumping. Ensure the black semicircle does not come into direct contact with skin while Elvie Pump is pumping
- Do not store wet or damp parts, as mold may develop
- Do not immerse the Hub or Charging Cable in water or other liquids. Wipe with a clean, damp cloth.
- This product contains small parts and a cable that may present a hazard. Keep out of reach of children.
- Only use Elvie manufactured parts with Elvie Pump, including the Charging Cable; do not substitute
- It is recommended to use an adapter compatible with both IEC 60601-1 and your regional power source
- Elvie Pump must only be used with chargers that are fully compliant with the relevant USB Specifications: USB 2.0 (5V, 500mA), USB 3.0/3.1 (5V, 900mA), USB Battery Charging (BC) 1.2 (5V, 1.5A). If you already have a charging adapter at home, we recommend checking:
  - The output voltage / current or power is in the range stated above
  - The charging adapter contains a country specific standards mark such as:

- The charging adapter is double insulated (Class 2 electrical certification)

- Do not drop or insert any foreign object into any openings on the Hub and always close the Charging Cap after charging
- Never operate an Elvie Pump if it has been damaged, is not working properly, or has been dropped into water. If damage is found, immediately discontinue use and contact our Customer Care team.

- If the package appears damaged, the tamper-proof sticker is removed and/or otherwise tampered with before use, do not use and contact our Customer Care team.
- Elvie Pump and its components are not heat-resistant. Keep away from heat sources
- Do not use flammable materials while using Elvie Pump or connecting Elvie Pump to a charger
- Elvie Pump should not be used while lying down or engaging in strenuous exercise or activities with risk of harm
- Do not use Elvie Pump while sleeping or overly drowsy
- Do not wear Elvie Pump when not pumping
- Do not attempt to reset Elvie Pump during use
- Do not use Elvie Pump when driving or operating heavy machinery
- Elvie Pump uses Bluetooth technology. Please follow airline rules for the use of portable electronic devices when flying. Furthermore, high altitudes, including while in a pressurized aircraft cabin, will affect the performance of Elvie Pump
- Always break the seal around your nipple before removing Elvie Pump from your breast
- Pumping can induce labor. Do not pump until after giving birth. If you become pregnant while breastfeeding or breast pumping, consult with a licensed healthcare professional before continuing
- Portable and mobile frequency communications equipment can affect the breast pump
- Only use Elvie Pump for its intended use as described in this manual
- Elvie Pump will not work when connected to the charger; always disconnect Elvie Pump from the charger and close the Charging Cap before use
- Always inspect all parts before use for damage and cleanliness
- Always ensure all parts are completely dry before assembling
- Do not place the Hub in the fridge or freezer
- Using a breast pump should not cause pain. Do not try to express with an intensity setting that is too high or uncomfortable. If excessive discomfort or pain is felt during pumping, stop pumping, break the seal around your nipple and discontinue the session
- Always ensure you are using the right sized Breast Shield. For assistance with correct Breast Shield sizing and comfort, please see the *Pump with Elvie* app, section 5, or talk to a lactation consultant or breastfeeding specialist
- Contact your healthcare professional if you express minimal or no milk, or if expression is painful
- If your breast becomes swollen, hot and/or painful, please seek immediate advice from your healthcare professional
- Extra care is necessary when using Elvie Pump in the vicinity of children
- There are no internal serviceable parts The battery is not replaceable.
- Only use Elvie manufactured parts with Elvie Pump, including the Charging Cable; do not substitute. Use of accessories or cables other than those specified could result in irreversible damage to your Hub

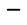
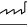
- At Elvie, we consider the security of our customers and systems a top priority. Our Security Research and Responsible Disclosure policy can be found here: <https://www.elvie.com/security-research-and-responsible-disclosure>
- Wash your hands thoroughly with soap and water before touching Elvie Pump, its components or your breasts, and avoid touching the inside of the Bottle or Lid. Only use drinking-quality tap water or bottled water for cleaning components that come into contact with milk
- Do not use harsh chemicals to clean the Breast Shield, Bottle, Valve, Spout or Seal
- If stored at minimum or maximum storage temperatures (see section 5.2), when removing from storage, please allow 24 hours for the pump to reach normal operating temperatures before use
- Do not expose Elvie Pump or its components to direct sunlight
- It is recommended that you keep your Elvie Pump up to date by periodically switching it on and connecting to the *Pump with Elvie* app when not in regular use

**Note: Please visit [elvie.com](https://www.elvie.com) for the full list of troubleshooting including alerts, warnings and factory reset for Elvie Pump**

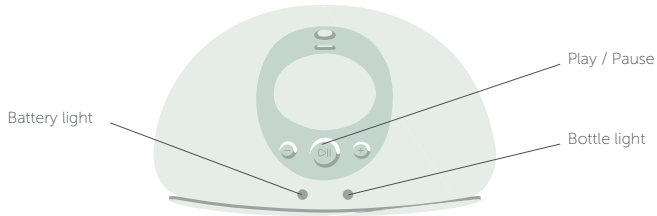
## 6.2 Symbols

	On/Off
	Increase intensity
	Decrease intensity
	Left and Right side selection
	Play/Pause
	Consult instructions for use
	Manufacturer
	Date of manufacture
	Do not dispose the device together with unsorted municipal waste (in accordance with local regulations)
	Compliance with international requirements for protection from electric shock. (Type BF applied parts)
	Protection against entrance of solid foreign objects and against harmful effects due to the entrance of water
	Manufacturer has taken all necessary measures to ensure that the product complies with applicable safety legislation for distribution within the European Community
	Serial Number
	Lot Number

## 6.3 Troubleshooting

Alerts and warnings on Elvie Pump

Play / Pause - White: slow flash	Pump paused
Battery Light - Green	Fully charged
Battery Light - Red	Battery low
Battery Light - Red: flashes then turns off	Battery empty
Battery Light - Orange	Charging
Bottle Light - White: Elvie Pump will pause	Milk bottle full
Play / Pause - White: flashes three times	Maximum / minimum intensity level reached
All lights – Flash then turn off	Technical error. Contact customer care

Please refer to [elvie.com](https://www.elvie.com) for the most up-to-date troubleshooting information.


If you are experiencing an issue with your Elvie Pump then you can contact us at: [www.elvie.com/contact](https://www.elvie.com/contact)

## 6.4 Factory reset

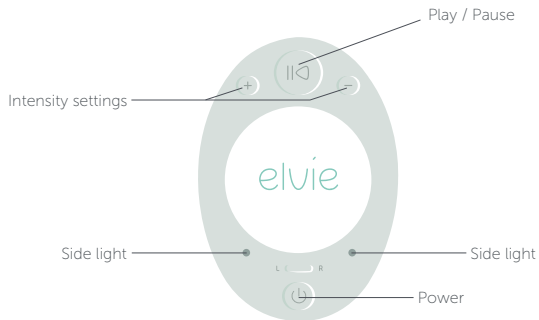To perform a factory reset:

1. Turn Elvie Pump on
2. Press and hold both   simultaneously for 5 seconds
3. Play/Pause and side LEDs will flash while Elvie Pump is resetting.  
The LEDs and Elvie Pump will then turn off

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2024 © Chiaro Technology Limited