Instructions for Use

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Mode d'emploi

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Bruksanvisning

Brugsanvisning

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Additional help and support

The Elvie Trainer app can be downloaded from the Apple App Store for iPhone, or Google Play for Android. If you need any help with your Elvie Trainer, please get in touch with our Customer Care team. Visit [elvie.com](https://www.elvie.com) for their details.

For Customer Care contact details and to see these instructions in other languages, visit [elvie.com/languages](https://www.elvie.com/languages)

Model: EL01 and EL02

1. Intended use and users

The Elvie Trainer is used to exercise and track pelvic floor muscle movement and tone.

Elvie Trainer is a single patient device, intended to be operated by adult females in a home environment.

Operating life

The operating life of Elvie Trainer is defined to be approximately 500 hours.

2. Contraindications, warnings and precautions

Contraindications

Do not use Elvie Trainer if you have given birth within the last six weeks.

Warnings

Only use Elvie Trainer for its intended use as described in this manual.

If the package appears damaged or otherwise tampered with before use, do not use and contact our Customer Care team using the details at the front of the guide.

This is a single-user product. Use by more than one person may present a health risk and voids the warranty.

This product contains a cable that may present a hazard. Keep out of reach of children.

When using electrical products, especially when children are present, basic safety precautions should always be maintained.

Elvie Trainer must only be used with the charging case, micro USB cable and custom sizing cover provided.

If you experience any of the following symptoms, stop using Elvie Trainer and consult a healthcare professional:

- You experience difficulty or persistent pain when inserting Elvie Trainer, even with water-based lubricant
- You experience abnormal bleeding
- Experience sudden urine retention or incontinence

Do not leave your Elvie Trainer inside your vagina for more than one hour at a time. We do not recommend exercising with Elvie Trainer for more than 15 minutes within a 24 hour period.

While there are no technical reasons not to use our product during pregnancy, we don't recommend using Elvie Trainer unless you have consulted with a licensed healthcare professional first.

It is recommended that you consult with a licensed healthcare professional before using Elvie Trainer if you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, implantable cardioverter defibrillator).

Do not use Elvie Trainer if you have an active urinary infection until symptoms have stopped. Use of Elvie Trainer with an active urinary tract infection may result in pain, tightness or reinfection.

No modification of this equipment is allowed. There are no service or maintenance activities required.

Precautions

If you have a medical condition affecting your pelvic health (for example, abnormal vaginal discharge, severe prolapse, are self-catheterizing, neurological conditions or experience frequent urinary infections), consult with a pelvic floor specialist before using Elvie Trainer.

Elvie Trainer is an exercise tracker for the pelvic floor. It must not be used to treat or prevent any medical conditions. If you are unsure whether or not you should use Elvie Trainer, please consult a licensed healthcare professional.

Elvie Trainer uses Bluetooth technology. Please follow airline rules for the use of portable electronic devices when flying.

Elvie Trainer must only be used with chargers that are fully compliant with the relevant USB Specifications: USB 2.0 (5V, 500mA), USB 3.0/3.1 (5V, 900mA), USB Battery Charging (BC) 1.2 (5V, 1.5A). If you already have a charging adapter at home, we recommend checking that the output voltage / current or power is in the range stated above. The charging adapter should also contain a country specific standards mark such as:

EU  US  CN 

Elvie Trainer operates within the 2.402-2.480GHz unlicensed ISM frequency band(s) and is limited to a maximum of 3dBm output power transmitted in the frequency band(s) in which the radio equipment operates.

Portable and mobile frequency communications equipment can affect the device.


It is recommended to use an adapter compatible with both IEC 60601-1 and your regional power source and that it is double insulated (Class II electrical certification):

3. What's inside

In the box you will find:

- 1. Pelvic Floor Trainer made with medical-grade silicone**
Use to track your pelvic floor exercises.
- 2. Charging case**
Keeps the Elvie Trainer safe and doubles up as a charger. Push to open.
- 3. Micro USB cable**
Connects the charging case to a USB power supply for charging.
- 4. Custom size cover made with medical-grade silicone**
Provides custom sizing for a better fit.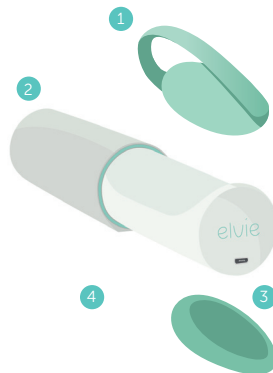

4. Charging Elvie Trainer

1. See above for details.
2. The charging case, with the Elvie Trainer inside, should be charged lying down lengthways.
3. Use the cable provided to connect the charging case to a USB power source, such as your computer or phone plug.
4. Check that the white light above the cable on the carry case turns on. The light will turn off when Elvie Trainer is fully charged.
5. The battery level is shown within the app.

Starting

Before first use, please remember to:

1. Charge Elvie Trainer

See previous page for details.

2. Get the Elvie Trainer app

Download the app through the App Store (for iOS) or Google play (for Android). Once installed, open the app and follow the on-screen instructions.

3. Clean Elvie Trainer

Please remember to wash your hands and Elvie Trainer with warm, soapy water before use, rinsing thoroughly.

4. Switch on Bluetooth on your phone

Go to your phone settings to turn on Bluetooth.

5. Using Elvie Trainer for the first time

Open the Elvie Trainer app to be guided on how to use Elvie Trainer for the first time

1. Connect Elvie Trainer

Elvie Trainer connects to your phone using Bluetooth. To connect your Elvie Trainer for the first time, simply squeeze it gently in your hand for 3 seconds.

2. Insert Elvie Trainer

Elvie Trainer is easy to insert. The pebble shaped part goes into your vagina and the tail remains outside, facing forward. Be sure to fully insert the pebble shaped part, but not too high up (images 1 and 2). If you pull down gently, you will feel slight resistance against the vaginal opening. If it feels uncomfortable to insert Elvie Trainer, try using some water-based lubricant.



3. Find the right fit

Many women prefer to use the optional cover to prevent Elvie Trainer falling out or moving about inside.

Try with and without the cover to see what feels best. Thread the Elvie Trainer tail through the small hole in the cover and pull over the Elvie Trainer (image 3). Always remember to take the cover off when charging.

4. Let's start by lying down

This relaxes the muscles in your thighs, buttocks and tummy, allowing you to focus on your pelvic floor muscles. Lie down with your legs slightly apart and hold your phone by your tummy. For the best results, start in a lying position for your isolated exercises and keep exercising in the same position. When you feel confident, you can try exercising while standing or sitting.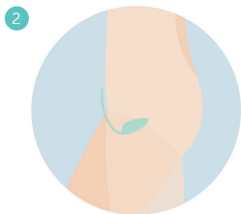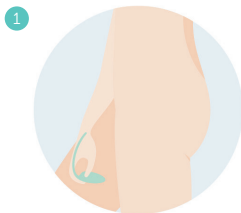

Optional Cover

6. How to exercise

The pelvic floor is a powerful little set of muscles that stretches like a hammock between your pubic bone and your tailbone. When healthy and toned, they lift up your organs for a strong core and great control.

If you aren't sure what it feels like to exercise your pelvic floor, imagine that you are trying to stop yourself peeing mid-flow. This muscle movement, lifting inwards and upwards, is what you need to master.

When exercising, remember to:

- Make sure your stomach, buttocks and thighs remain relaxed and try not to hold your breath
- Take a deep breath to relax and as you breathe out squeeze your pelvic floor inwards and upwards
- The Elvie Trainer app provides more guidance during your workouts. Tap the help icon in the top left hand corner for information on how to perfect your exercise technique

7. How to connect

Elvie Trainer works best when it is able to connect easily with your phone. This means not obscuring the tail too much (for example by crossing your legs). Try to keep your legs open when exercising and your phone in the line of sight with Elvie Trainer.

At the beginning of each exercise gently squeeze your Elvie Trainer so that it starts searching for your phone. It can take up to 30 seconds to connect.

If Elvie Trainer disconnects during an exercise it should reconnect automatically. If it doesn't, please follow the on-screen instructions remembering to squeeze Elvie Trainer for at least 3 seconds when trying to connect.

8. Taking care of Elvie Trainer

Cleaning

Elvie Trainer is not provided sterile, so make sure you clean Elvie Trainer before and after every use with warm, soapy water. Remember to rinse thoroughly. The charger carry case can be wiped clean with a damp cloth.

It is important Elvie Trainer is completely dry before storing in the charging case.

Don't:

1. Wash Elvie Trainer in boiling water
2. Use harsh cleaning agents
3. Clean Elvie Trainer in a sterilizer, microwave dishwasher or washing machine
4. Leave Elvie Trainer trapped under anything
5. Use Elvie Trainer in the bath

Storing


Keep Elvie Trainer clean, dry and snug in its carry case. Make sure the tail is on top and facing outwards as per the diagram below.

Things to avoid

- Extreme heat
- Extreme cold
- Sustained pressure

Transport and Storage conditions -20°C to +60°C / 15 -90% RH / ambient pressure 70 - 106kPa.

Operating conditions +5°C to +40°C / 15 - 90% RH / ambient pressure 70 - 106kPa

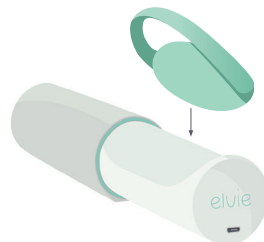9. Disposal

Elvie Trainer contains a Li-ion rechargeable battery. Please dispose of it at a designated collection point for the recycling of waste electrical and electronic equipment. The appropriate disposal of Elvie Trainer will help conserve natural resources and ensure that it is recycled in a manner that protects human health and the environment. For more information about where you can drop off your waste for recycling, please contact your local authority or where you purchased your product.

10. Troubleshooting

For all FAQs see [elvie.com/faqs/elvie-trainer](https://www.elvie.com/faqs/elvie-trainer)

11. Certification

Elvie Trainer:

IC: 20167-EL0115
FCC: 2AEHI-EL0115
CMIIT ID: 2020DJ7847

Charging case:

IC: 20167-EL0215
FCC: 2AEHI-EL0215
CMIIT ID: 2020DJ7847

IP Rating:

Elvie Trainer IPX7
Charging case IPX2

Electromagnetic compatibility (EMC) advice:

This device complies with part 15 of the FCC rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct interference by one or more of the following measures:

- i) Reorient or relocate the receiving antenna
- ii) Increase the separation between the equipment and receiver
- iii) Connect the equipment into an outlet on a circuit different from that to which the receiver is connected
- iv) Consult the dealer or an experienced radio/TV technician for help

FCC Radiation Exposure Statement: This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Hereby, Chiaro Technology Ltd declares that the radio equipment type BF is in compliance with Directive 2014/53/EU.

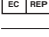
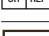
The full text of the EU declaration of conformity is available at the following internet address:

elvie.com/certification

12. Technical specification

Certification markings can be found under the Elvie Trainer tail and inside the charging case.

Symbols	Description
	Manufacturer
	Date of Manufacture

Symbols	Description
	Serial number
	Lot number
	Battery Charger system in accordance with Title 20 California Appliance Efficiency Regulation
	Compliance with international requirements for protection from electric shock. (Type BF applied parts) from electric shock.
	Refer to the instructions for use when operating the device.
	To identify equipment meeting the safety requirements specified for Class II equipment according to IEC 61140.
	Waste Electrical and Electronic Equipment do not dispose of this product in unsorted municipal waste stream.
	CE marking, indicates EU conformity
	Regulatory Compliance Mark, indicates compliance with ANZ radiocommunications requirements.
	United Kingdom Conformity Assessed
	Indicated compliance with Japan Radio Law
	European Authorized Representative
	Swiss Authorised Representative
	An indication that the device is a medical device

Symbols	Description
	Protection against entrance of solid foreign objects and against harmful effects due to moisture as defined in international standard EN 60529 <ul style="list-style-type: none"> • IPX2: No protection from solid objects. Protected against vertically falling water drops when enclosure tilted up to 15° • IPX7: No protection from solid objects. Protected against the effects of temporary immersion of water.
	

For all certification and approval, visit elvie.com/certification

13. Warranty

Elvie Trainer is backed by a limited manufacturer's warranty against any defects in materials and/or workmanship for two (2) years.

For the complete terms and warranty information, please visit elvie.com/warranty or contact Customer Care. This policy is offered in addition to your legal rights.

This is a single-user product. Use by more than one person may present a health risk and voids the warranty. This product may only be returned for a refund within 30 days of purchase provided it is new, unopened, unused and in its original packaging with the manufacturer's factory seal still intact.

Please keep your proof of purchase in the event that you need to contact Customer Care.

Residents in the United States: to request a hard copy of the US limited warranty information, call us at 808 300-9989, or write us at Elvie - Chiaro Technology, Office Manager, 184 7th Ave, Unit #107, New York, NY 10011.

If you experience any serious incident that has occurred in relation to this product, please report it to Elvie. If you are in the EU, please report it to the competent authority of your Member State.

Please keep a copy of these instructions for future reference.

For further questions or issues, please refer to the help section on the Elvie Trainer app or drop us an email: questions@elvie.com

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© 2024 Chiaro Technology Limited